



HAIR COLLECTION FORM
I GENE LLC

Company: _____
Full name: _____

Contact phone: _____
E-mail: _____
Date of hair sampling: _____

-----Fold line-----

PLEASE TYPE HAIR, AS SHOWN

Name: Reg. numb: Birth date:

-----Fold line-----

IMPORTANT NOTE. Responsibility for the correspondence of biological material to the marking on the form the customer takes care of himself. Dirty (contaminated) hair samples will not be accepted for testing.

Hair collection guidelines:

- Hairs should be performed in medical sterile gloves.
- Provide the necessary information about the animal.
- Pull **at least 40 hairs** from a clean/dry tail with visible hair roots (follicles).
- Position all hair roots/follicles together at one end. Do not touch the hair follicles.
- Secure with adhesive tape wrapped on the shaft approximately 2,5 cm from the hair roots / follicles
- Attach sample to the application above. Tape in two places to avoid unsticking during transportation (adhesive tape should not touch follicles). The tail of the hair sample without follicles, protruding beyond the edge of the form page, can be cut off.
- Fold the form page over the fold lines immediately after attaching the hair sample. The attached hair sample must be covered with the edges of the form page. Put the folded sheet into an envelope and seal it to avoid mixing samples.
- **Each hair sample must be attached to a separate form. Form should be placed in a separate envelope to prevent mixing of the samples.**
- Form fields "Registration number" and "Birth date" must be filled for each animal.
Samples without this information will not be accepted for testing.